


Doffing guide for X-plore 8000

Do NOT remove the respirator hood until you have vacated the contaminated area.



1. Using the fitting loops located under chin, pull hood up and over your head.

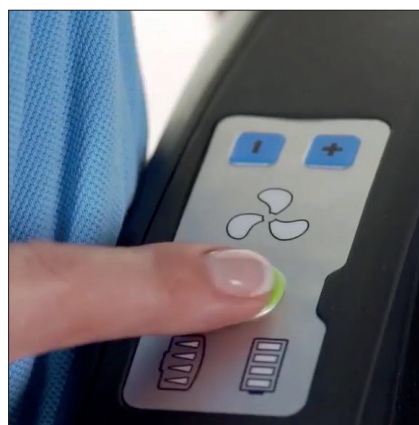


2. Hold the power button down for two seconds to turn off the device.



3. Detach hood from hose. Dispose of hood if single use.



4. Detach hose from blower unit. Remove and dispose single-use hose cover if used.

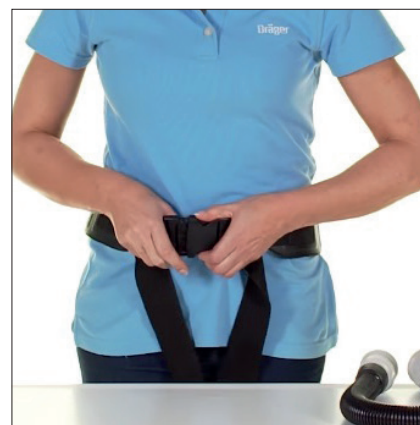

5. Unfasten the belt and remove

Wash your hands thoroughly before and after doffing. Your unit should be cleaned and stored according to the manufacturer's instructions.