

Assembly guide for X-plore 8000

Always visually inspect parts before assembling. This guide presumes that filters are new, batteries are charged, and that parts are clean and dry.



1. Insert a fully charged battery into the blower unit.



2. Remove the filter splashguard by pressing down on the button.



3. Place a P3 filter into the lid, pressing down to 'click' into place.



4. Fold the lid at an angle into the blower unit, clicking into place.



5. Align arrows on the belt and blower unit to match the orientation.



6. Tilt the connecting plate at an angle and slide into place.



7. Press down on the connecting plate until you hear a 'click'.

Do not enter the contaminated area until your X-Plore 8000 powered air purifying respirator unit has been donned.